



08186 Llíssà d'Amunt
Barcelona
Spain

Tel.:+ 34 93 860 90 00
Fax:+ 34 93 860 90 17
e-mail: biokit@biokit.com
www.biokit.com

Date: 19/02/05

Ref: 173/MKT/E/55

SUBJECT **Best 2000 – Sample Rack**



Attached to this letter you will find instructions to readjust the spring clip fingers of the Best 2000 sample rack to avoid the sample tube movement.

The manufacturer will start shipping product with the clips already pulled out for the standard sample tube.

We extremely recommend reading carefully the instructions before proceeding.

Please, do not hesitate to contact us if you need further details.

Adjusting Spring Clips in the DSX Sample Racks

1. Using a pair of needle nose pliers grip the back of the rack and spring clip, see figure 1.

This will enable you to pry the legs of the clip without putting undo stress on the back of the rack.

2. Using a second pair of needle nose pliers grab a leg of the spring clip and pull forward see figure 2. Repeat for the second leg.
3. When finished both legs should be close to crossing or may cross a little bit. You want to pull out the legs as much as you need to ensure the sample tube is not too loose in the rack.

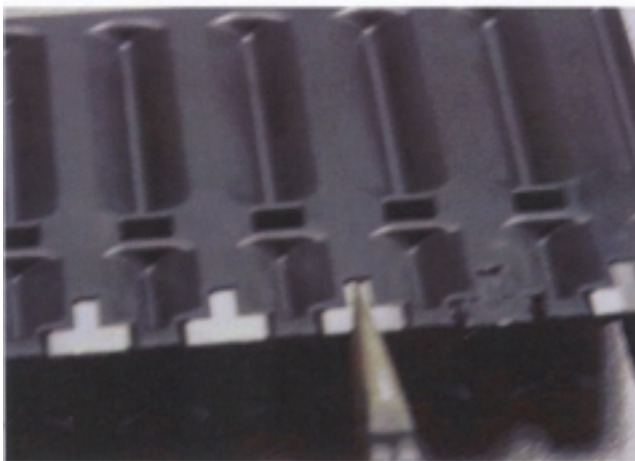


Figure 1 (Rear View of Sample Rack and Springs)

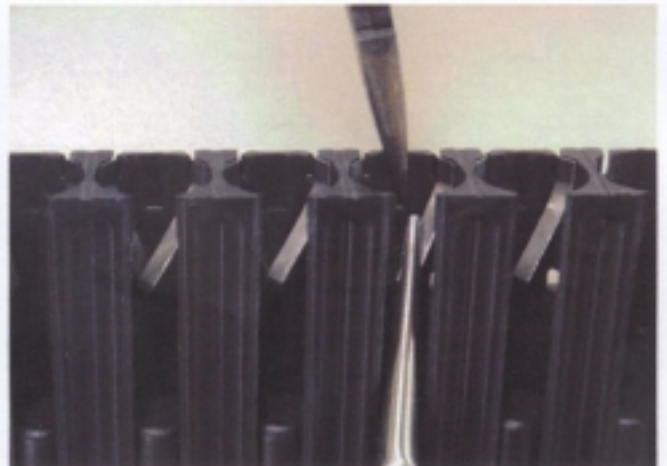


Figure 2 (Holding the back of the Rack as you pull out the Spring Legs)